



# ALTA DIAGNOSTICS, INC.

2560 BUSINESS PARKWAY Ste. C, MINDEN, NV 89423  
 (800) 359-9691 – (775) 267-3001 – FAX (775) 267-1142

OPEN VIAL  
 STABILITY  
 18 Months

## Semiquantitative Values for LIQUID URINE DIPSTICK CONTROL

Lot #025033 NEGATIVE  
 Expires January 2012

Lot #041034 POSITIVE  
 Expires February 2012

Values range for CLINITEK™, YELLOW IRIS™, and VISUAL readings.

SIEMENS MULTISTIX 10 SG	NEGATIVE	POSITIVE	
GLUCOSE	NEGATIVE	100(TR) ≥ 1000 mg/dl	g/dl% 1/10(tr) ≥ 1
BILIRUBIN	NEGATIVE	SM - LRG	+ - +++
KETONES	NEGATIVE	5 ≥ 160 mg/dl	Tr ≥ Lg
SPECIFIC GRAVITY	1.015 ≥ 1.030	1.005 – 1.020	
BLOOD	NEGATIVE	TR - LRG	TR - +++
PH	5.0 – 7.0	6.0 – 8.5	
PROTEIN	NEGATIVE	30 - ≥ 300 mg/dl	+ ≥ +++
UROBILINOGEN	NORMAL / 0.2 mg/dl	1 ≥ 8 MG/DL	
NITRITE	NEGATIVE	POSITIVE	
LEUKOCYTES	NEGATIVE	TR - LRG	Tr - +++

SIEMENS MULTISTIX PRO	NEGATIVE	POSITIVE	
GLUCOSE	NEGATIVE	100(TR) ≥ 1000 mg/dl	g/dl% 1/10(tr) ≥ 1
KETONES	NEGATIVE	5 ≥ 160 mg/dl	Tr ≥ Lg
SPECIFIC GRAVITY	1.015 ≥ 1.030	1.015 ≥ 1.030	
BLOOD	NEGATIVE	TR - LRG	TR - +++
PH	5.0 – 7.0	6.0 – 8.5	
NITRITE	NEGATIVE	POSITIVE	
LEUKOCYTES	NEGATIVE	TR - LRG	Tr - +++
PROTEIN	NEGATIVE	30 - ≥ 300 mg/dl	+ ≥ +++
CREATININE	10-100 mg/dl   0.9-8.8 mmol/L	10-100 mg/dl	0.9-8.8 mmol/L
PRO./CREAT. Ratio	<30 NORMAL	>300 ABNORMAL	
ALB./CREAT. Ratio	Alb 10/Creat10/50-Ratio < 30	Alb 150/Creat 10/50-Ratio > 300	
REFRACTOMETER	1.016 – 1.026	1.009 – 1.019	
PROTEIN-SULFOSAL	NEGATIVE	≥ 100 mg/dl	
HCG	NEGATIVE	POSITIVE	

SIEMENS REAGENT TABLETS	NEGATIVE	POSITIVE
ACETEST™	NEGATIVE	SM - LRG
CLINITEST™	NEGATIVE	¼% - 1%
ICTOTEST™	NEGATIVE	POSITIVE

PACKAGED 6X20ML (DROPPER VIALS)

50-60 USES PER VIAL



OVER

3/18/10

This URINE DIPSTICK CONTROL may be used with any of the Siemens instruments or Chemistry Reagent Tests  
 Check [altadiagnostics.com](http://altadiagnostics.com) for insert updates

CHEMSTRIP™	NEGATIVE	POSITIVE
SPECIFIC GRAVITY†	1.010 – 1.020	≤ 1.005 - 1.015
pH	5.0 – 7.0	6.0 – 9.0
LEUKOCYTES	NEGATIVE	Tr - ++ / 25 – 500 LEU/ML
NITRITE	NEGATIVE	POSITIVE
PROTEIN	NEGATIVE	Tr - +++ / 15 - 500 MG/DL
GLUCOSE	NORMAL	Tr - ++++/50 - ≥ 1000 MG/DL
KETONES	NEGATIVE	+ - +++/15 - ≥ 150 MG/DL
UROBILINOGEN	NORMAL	1 - ≥ 8 MG/DL
BILIRUBIN	NEGATIVE	+ - +++/1 - 6 MG/DL
BLOOD	NEGATIVE	Tr - ++++/10 - 250 ERY/uL
HCG	NEGATIVE	POSITIVE

## Value range for QUIDEL QuickVue UrinChek™

QuickVue	NEGATIVE	POSITIVE
BILIRUBIN	NEGATIVE	+ - +++ / 1 – 6 mg/dl
UROBILINOGEN	NORM	+ - +++ / 1 - ≥ 8 mg/dl
KETONES	NEGATIVE	+ - +++ / 15 – 150 mg/dl
ASCORBIC ACID	NEGATIVE	NA
GLUCOSE	NEGATIVE	+ - +++ / 50 – 300 mg/dl
PROTEIN	NEGATIVE	+ - ++ / 15 – 100 mg/dl
BLOOD	NEGATIVE	+ - +++ / 10 –250 ery/L
pH	5.0 – 7.0	6.0 – 9.0
NITRITE	NEGATIVE	+ / POSITIVE
LEUKOCYTES	NEGATIVE	+ - ++ / 25 – 500 wbc/ul
SPECIFIC GRAVITY	1.010 – 1.020	≤ 1.005 – 1.015
HCG	NEGATIVE	+ / POSITIVE

† NOTE: Please see REFRACTOMETER values on front page.

- Remove** control from the refrigerator and allow to come to room temperature. Control may be stored in a common use refrigerator (refrigerator at the Nurse's Station). Control may also be left at room temperature and will remain stable for up to 8 weeks when stored away from ultraviolet light.
- Remove** dropper bottle cap, invert (**do not shake**) and apply control material directly onto the dipstick. With the tip of the bottle touching the dipstick, draw the tip across all of the reagent pads, while gently squeezing the sides of the dropper bottle. Excess control material may be removed by tilting the dipstick on edge and resting on a paper towel.