



ALTA DIAGNOSTICS, INC.

3123 Research Way Ste 214, Carson City, NV 89706
 (800) 359-9691 (775) 283-5780 FAX: (775) 283-5787

**18 MONTH
OPEN VIAL
STABILITY**

LIQUID URINE CONTROL

FOR

MICROALBUMIN

Level 1: 202221
Expires July 2014

Level 2: 202222
Expires July 2014

	QUANTITATIVE ASSAY		MICRAL CHEMSTRIP*		PROCEDURE
	Level 1	Level 2	Level 1	Level 2	
UNITS	Mg/L	Mg/L	Mg/L	Mg/L	<ol style="list-style-type: none"> For dipstick use, remove cap and insert dipstick. Remove dipstick after the time recommended by the dipstick manufacturer. Follow the procedure for testing as described by the manufacturer of the dipstick. If the vial will be used within 30 days you may recap the control and leave it at ambient temperature (15-25 degrees C). IF The bottle is used beyond 30 days, store the bottle at 2-8 degrees C. Any changes in the reagents, methods or instrument methodology by the manufacturer may result in different values. <p>Consult manufacturer's instructions for the procedure for further information.</p> <p>* Chemstrip is a registered product of Roche.</p>
RANGE	6 ± 5.5	40 ± 20	0 - 20	20 - 100	
DAY 1					
DAY 2					
DAY 3					
DAY 4					
DAY 5					
DAY 6					
DAY 7					
DAY 8					
DAY 9					
DAY 10					
DAY 11					
DAY 12					
DAY 13					
DAY 14					
DAY 15					
DAY 16					
DAY 17					
DAY 18					
DAY 19					
DAY 20					
DAY 21					
DAY 22					
DAY 23					
DAY 24					
DAY 25					
DAY 26					
DAY 27					
DAY 28					
DAY 29					
DAY 30					
DAY 31					
TOTAL					
MEAN					